

Hair Collection Instructions

 Pull (do not cut) hair from the tail switch, not tail head.



3. Write the animal's ID number on the collector in the space provided.



Insert the root end of the hair in the middle of the collector. Close the collector by sealing the plastic cover over the hair and then insert the tab into the slot.



Make sure hair is free of fecal material and dirt. Roots must be clearly visible. Approximately 20 to 30 hair roots are needed.



4. Open the collector and peel back the plastic cover.



Trim the excess hair extending from the collector. Put all collectors in the plastic specimen bag for shipment.



To place an order, or for more information, contact your Igenity representative, or visit our website at www.neogeneurope.com Igenity a Neogen Company • The Dairy School • Auchincruive, Ayr, Scotland • KA6 5HU
Tel:0845 603 8895 • www.neogeneurope.com • igenity@neogeneurope.com